

SIGVARIS  
GROUP

PRODUCT LOOKBOOK  
Medically Complex Edema Solutions



Upper & Lower

sigvaris

Move your  
life with  
**WrapAbility**



sigvaris

## Compreflex®

20-50mmHg



### Accurate, measured therapeutic compression for self-managed care

The Compreflex series of products includes low profile compression devices that deliver mild to moderate compression to lower extremities. Each Compreflex device is made of soft, conforming inelastic material, offering low to minimum stretch for proper fit and gradient compression. Key features include our patented Accutab® Devices which ensure the accurate application of the prescribed dosage of therapeutic compression, an interior stretch panel for easy donning, and opposing straps for a comfortable fit.

#### Ideal for

- Chronic Venous Disease
- Chronic Venous Insufficiency (CVI)
- Wound Management including, but not limited to ulcers due to venous stasis
- Non-specific or post-surgical edema
- Lymphedema - Active and Maintenance



#### Accutab® Devices

The Accutab accurately delivers the level of resting compression prescribed by the health care provider. An easy-to-use device, the Accutab ensures the safe delivery of compression in doses of 20-30mmHg, 30-40mmHg, or 40-50mmHg.



For a PDF of the product order form, scan the QR code or visit [sigvaris.group/mceforms](https://sigvaris.group/mceforms)

## Compreflex<sup>®</sup> Transition

### CALF

The Compreflex Transition Calf includes the inelastic calf wrap and a pair of Transition Liners. The Transition Liner offers 10–15mmHg of compression in the foot and ankle. The Compreflex Transition Calf is ideal for patients with active venous ulcers. It includes a set of accutab devices.



## Compreflex<sup>®</sup> Complete

### CALF

The Compreflex Complete Calf includes the inelastic calf wrap and a pair of Complete Liners. The Complete Liner offers 10–15mmHg of compression in the foot, ankle, and calf. The Compreflex Complete Calf is ideal for patients with active venous ulcers. It includes a set of accutab devices.



## Compreflex<sup>®</sup> Standard

### CALF

The Compreflex Standard Calf includes the inelastic calf wrap and a pair of non-compressive Basic Liners. The Compreflex Standard Calf is ideal for patients with chronic edema without foot involvement.



## Compreflex<sup>®</sup> Standard

### CALF & FOOT

The Compreflex Standard Calf and Foot offers compression in the foot, ankle, and calf. The inelastic calf wrap is paired with the Compreboot Standard Foot, a set of Accutab Devices, and the non-compressive Basic Liner. When combined, the products create a complete compression system for the lower leg.



## Compreflex<sup>®</sup> Standard

### THIGH

The Compreflex Standard Thigh is an inelastic compression device for the thigh. The product includes a set of Accutab Devices and a Belt and Hip attachment to support the product throughout daily use.



## Compreflex<sup>®</sup> Standard

### KNEE

The Compreflex Standard Knee is an inelastic compression device for the knee. The product includes a set of Accutab Devices and a Butterfly Strap for added compression to the popliteal or patella regions.



## COMPREFLEX<sup>®</sup> STRAP EXTENDER

- Adds 10cm of adjustable range to a single strap
- 1 set extends 1 strap
- Strap extender works only with the Compreflex brand of products

Color
Black
Beige




## ACCUTAB<sup>®</sup> DEVICES

Compression measuring device design for use with the Compreflex line of inelastic garments for the lower extremity.

The Accutab accurately delivers the level of compression prescribed by the health care provider. An easy to use device, the Accutab ensures the safe delivery of compression in doses of 20–30mmHg, 30–40mmHg, or 40–50mmHg.

Size
Small
Medium
Large
X Large
XX Large

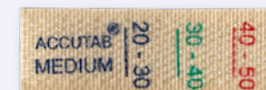

One set includes five Accutabs.

## GRIPPER PATCHES

- Adds traction and grip to foot compression garments
- Hook and loop bottom makes patches easy to apply
- 1 set includes two patches

Color
Black



For a PDF of the product order form, scan the QR code or visit [sigvaris.group/mceforms](http://sigvaris.group/mceforms)

Step into your  
life with  
**WrapAbility**



## Coolflex®

20–50mmHg



### Lightweight, breathable compression wear

Coolflex is a lightweight compression device optimal for individuals with fragile or broken skin. The patented EZ-Close® pressure system closes laterally, promoting independence and helping to improve patient outcomes. Made of a non-stretch mesh material, the Coolflex transfers heat and wicks away moisture, leaving the skin cool and dry.

The Coolflex Standard Calf includes the inelastic calf wrap and a pair of non-compressive basic liners.

### Ideal for

- Chronic Venous Disease
- Chronic Venous Insufficiency (CVI)
- Wound management including, but not limited to ulcers due to venous stasis
- Lymphedema
- Non-specific or post-surgical edema



### Accurate compression is easy with Coolflex Standard Calf

The integrated EZ-Close® Pressure System provides visual indicators for 20–30, 30–40, and 30–50mmHg on each tab.

\* Option to order Coolflex with Transition Liner if mild foot compression is required



For a PDF of the product order form, scan the QR code or visit [sigvaris.group/mceforms](https://sigvaris.group/mceforms)

# Foot Compression Wear

## COMPREBOOT® LITE FOOT

Variable Compression



- Ultra-thin profile that easily fits into most footwear
- Highly adjustable design may be trimmed to fit
- Provides mild, therapeutic compression



## COMPREBOOT® STANDARD FOOT

Variable Compression



- Four-way stretch for firm compression around ankle
- Ideal for mild to moderate chronic edema
- Low stretch material conforms to foot shape



## COMPREBOOT® PLUS FOOT

Variable Compression



- Chip foam lining creates high/low pressure ridges
- Removable gripper patches provide traction
- Provides moderate, therapeutic compression



## COOLFLEX® STANDARD FOOT

Variable Compression



- Low profile design allows boot to fit in most footwear
- Spacer Fabric is highly breathable for cool-wearing comfort
- Durable metal clasps for closure



# Liner

## COMPLETE LINER

10-15mmHg in foot, ankle, and calf

- Comfortable knee-high liners
- Extra thick sole from toe to heel for added comfort



## TRANSITION LINER

10-15mmHg in foot and ankle

- Soft, flat seam knee-high liners
- Extra thick sole from toe to heel for added comfort



## BASIC LINER

Non-compressive

- Non-compressive, protective, moisture-wicking liner
- Smooth toe seam for added comfort





# Compreflex® Reduce

Variable Compression

## Complete Decongestive Therapy (CDT) Aid in 3 easy steps

The Sigvaris Compreflex Reduce is an innovative compression system used during the decongestive phase of lymphedema therapy. A replacement for tedious traditional bandage wrapping, the Compreflex Reduce is easy to size, customizable at the site of care, simple to assemble, and conveniently adjustable during daily wear.

The Compreflex Reduce garment promotes patient independence and encourages adherence to a planned treatment protocol. Key features include an interior stretch panel which holds the garment in place and Tension Tabs, which are visual indicators that clearly show where to position straps. Each garment may be combined with foam or other padding materials to promote or padding materials to promote gradience and maximize lymphatic drainage.

The Compreflex Reduce line of products includes garments for the foot, calf, knee, and thigh.

### Ideal for

- Lymphedema
- Patients in the decongestive phase of lymphedema treatment



### Tension Tab™

For accurate and consistent application of tension, the Compreflex Reduce garments include Tension Tabs. Tension Tabs are indicators that visually mark the appropriate tension levels as prescribed by the therapist. The Tension Tabs allow for easy readjustment of the garment for clinically effective compression all day long.



For a PDF of the product order form, scan the QR code or visit [sigvaris.group/mceforms](https://sigvaris.group/mceforms)



## Compreflex® Reduce **CALF**

The Compreflex Reduce Calf provides firm gradient compression to the calf. Pair the Compreflex Reduce Calf with the Compreflex Reduce Foot for complete below knee compression.



## Compreflex® Reduce **THIGH**

The Compreflex Reduce Thigh provides firm gradient compression to the thigh. The Belt and Hip attachment is included to help secure the thigh garment in place. Pair the Compreflex Reduce Thigh with the Compreflex Reduce Knee for upper leg compression.



## Compreflex® Reduce **KNEE**

The Compreflex Reduce Knee provides firm gradient compression to the knee. Pair the Compreflex Reduce Knee with the Compreflex Reduce Foot, Calf, and Thigh for full leg compression.



## Compreflex® Reduce **FOOT**

The Compreflex Reduce Foot provides firm gradient compression to the foot and ankle. Pair the Compreflex Reduce Foot with the Compreflex Reduce Calf for complete below knee compression.



## Compreflex® Reduce **LOBE SLING**

Variable Compression



- Intended for use with the Compreflex Reduce Foot, Calf, and Thigh
- Available for the lower extremity
- Completely customizable
- Lifts and compresses areas exposed outside of the garment
- Pairs well with our Therapy Supplies, Grayfoam, Wavefoam, Wavefoam II to support the patient during treatment

### Ideal for

- Lymphedema
- Edema
- Chronic Venous Insufficiency (CVI)



Range 3 shown at left.  
Range 1 not shown  
(contains two fewer straps and tabs)



# Medaform®

30-40mmHg




## Durable, high grade non-stretch compression

Sigvaris Medaform garments are for individuals in the advanced stages of lymphedema. Patients with Stage II or III Lymphedema benefit from the pressure ridges the inner Wavefoam liner creates. These high/low pressure ridges direct the lymph fluid and promote drainage. The straps and inner foam liner of the Medaform garment are replaceable, offering a durable, long-term solution for the treatment and maintenance of lymphedema.

### Ideal for

- Lymphedema
- Fibrotic Tissue

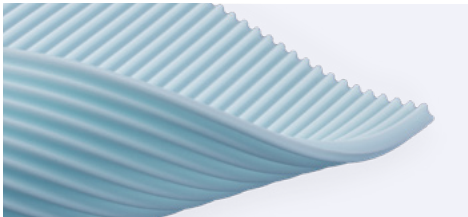

### Wavefoam Padding

The inner foam liner has high/low pressure ridges to help facilitate lymph movement.



## Medaform® Standard ARM

The Medaform Standard Arm provides firm gradient compression from the hand to the axilla. The Medaform Standard Arm includes the Medahand. The Medahand provides comfortable, moderate compression for the hand.



## Medaform® Standard CALF & FOOT

The Medaform Calf and Foot garment provides firm gradient compression from the foot to the knee. The Medaform calf garment is paired with the Compreboot Plus for added foot compression.



## Medaform® Full LEG

The Medaform Full Leg garment provides firm gradient compression from the foot to the gluteal fold. The garment provides full coverage of the foot, ankle, calf, knee, and thigh. The Medaform Full Leg is paired with the Compreboot Plus Foot for compression in the foot.



For a PDF of the product order form, scan the QR code or visit [sigvaris.group/mceforms](https://sigvaris.group/mceforms)



# Chipsleeve®

15-20mmHg\*




**Soft, massaging garment that promotes lymph fluid movement and drainage**

Sigvaris Chipsleeve garments are made for the upper and lower extremity. Filled with chip foam, each padded product is designed with directional channels, which work to efficiently direct lymph fluid and promote drainage. Patients with fibrotic limbs will enjoy the comfortable fit and massaging effect of the chip-filled garment.

Each Chipsleeve garment is made of four-way stretch fabric. The fabric contains moisture-wicking properties and is bacteriostatic to help control odor.

### Ideal for

- Lymphedema
- Fibrotic Tissue

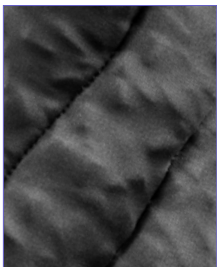

### Foam Chips

Open-cell foam chips massage superficial lymphatics

### Directional Channels

Channel formations facilitate efficient lymph drainage



\*When combined with the oversleeve

## Chipsleeve® Standard CALF & FOOT

The Chipsleeve Standard Calf and Foot garment provides mild to moderate gradient compression from the foot to knee. The garment is a single piece that easily glides onto the limb. Includes a black oversleeve for additional compression.



Black oversleeve not shown.

## Chipsleeve® Standard ARM


The Chipsleeve Standard Arm provides mild to moderate gradient compression from the hand to axilla. Includes a black oversleeve for additional compression.



Black oversleeve not shown.

## Chipsleeve® Full LEG

The Chipsleeve Full Leg garment provides mild to moderate gradient compression from the foot to the gluteal fold. Includes a black oversleeve for additional compression.

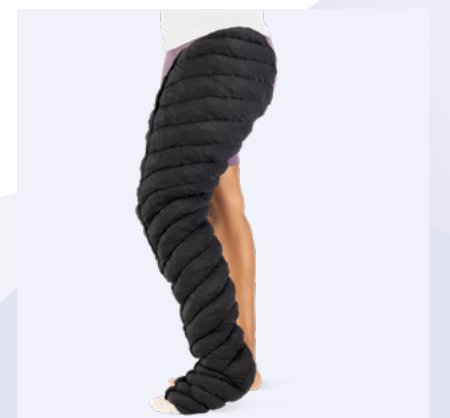

For a PDF of the product order form, scan the QR code or visit [sigvaris.group/mceforms](https://sigvaris.group/mceforms)

# Comprevest®

15-25mmHg



The Comprevest offers mild to moderate compression to the chest and back. Designed for an improved fit and overall comfort, the Comprevest includes wide hook and loop straps and adjustable hook closures. The Comprevest is offered in sizes Small to XX-Large with cup sizes A/B – DD.

For added stimulation of the affected area, pair the Comprevest with the Chip Pad Bra. With chip foam sewn into channels, the Chip Pad Bra massages and breaks down fibrotic tissue, while the channeling directs the fluid for removal. The Chip Pad Bra is designed to be inserted into the interior side pocket of the Comprevest.

## Ideal for

- Post-surgical procedures including mastectomy, lumpectomy or radical reconstruction
- Lymphedema

*Anterior hook adjustment for optimal fit*

*Easy-to-adjust straps*

*Interior pockets hold Chip Pad Bra or prosthetic insert*



# Neck and Mandible Pad

The Neck and Mandible Pad is made from soft materials that conform to the mandible and neck. The Neck and Mandible Pad provides mild compression and uses easily adjustable straps that secure the garment in place.



# Compreshort®

## SHORT & CAPRI

10-15mmHg



The Compreshort offers mild, supportive compression for the treatment of genital, abdominal, and upper thigh edema. Patients with lymphedema, edema, and lipedema benefit from the therapeutic compression and support this garment provides. Available in short and capri lengths.

## Ideal for

- Lymphedema
- Lipedema



Product available in short, capri, and custom lengths.

# Compreshape™

## SHORT & CAPRI


10-15mmHg



Designed as outerwear, the Compreshape offers mild supportive compression. The stylish waistband provides abdominal coverage and the smooth seams give an attractive appearance.

## Ideal for

- Lymphedema
- Lipedema

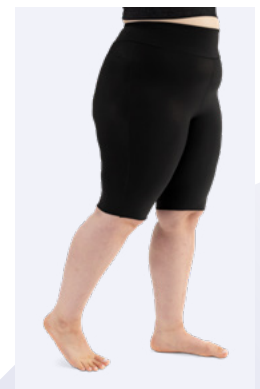

Product available in short and capri lengths.

*Smooth seams create stylish finish for outerwear design*



For a PDF of the product order form, scan the QR code or visit [sigvaris.group/mceforms](https://sigvaris.group/mceforms)

# Hand Compression Wear

## MEDAHAND®

Variable Compression



- Open-cell foam chips to gently massage tissue
- Conforming design to ensure proper fit and gradient compression
- Breathable, moisture-wicking, anti-odor, and soil-release fabric
- Effective as a compression garment or bandage liner



## MEDAGLOVE®

Variable Compression



- Conforming design with finger separation ensures proper fit
- Open-cell foam chips gently massage tissue
- Gradient compression from fingers to wrist
- Effective as a compression garment or bandage liner
- Breathable, moisture-wicking, anti-odor, and soil-release fabric



## DORSAL POCKET GLOVE™

Variable Compression



- Ultra-thin profile for full hand function during use
- Full finger compression with finger loops for easy application and removal
- Dorsal pocket holds Chip Pad Dorsum (sold separately) or other therapeutic insert
- Palm pad provides soft, mild compression



Define your  
story with  
**WrapAbility**



Live your life. Be yourself.

sigvaris

# Custom Garments

Sigvaris offers a line of custom compression garments for patients who measure outside of the standard size range of the ready-to-wear garments or those with a complex limb shape.



## Chipvest®


Variable Compression



The Chipvest contains open-cell foam chips that help to soften fibrotic areas. The chip foam is formed into channels to help direct lymph fluid and to facilitate drainage. To improve fit and comfort, the garment includes a back drawstring and corset design as well as hook and loop shoulder straps that are easily adjusted by the patient. Clinicians may further customize the product by selecting the location of the foam (unilateral or bilateral).

### Ideal for

- Lymphedema
- Post-surgical or non-specific edema

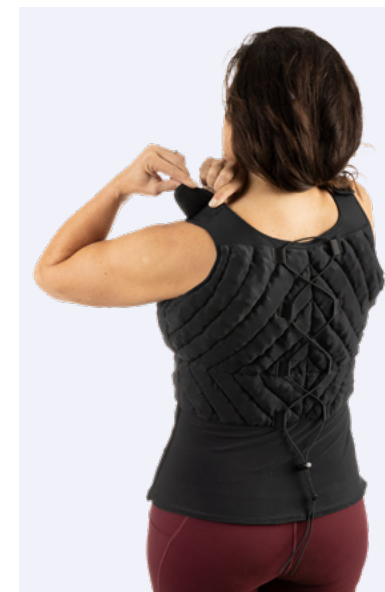

## Chipsleeve® Custom ARM

15-25mmHg\*



The Chipsleeve Custom Arm provides mild to moderate gradient compression from the hand to the axilla.

The Chipsleeve Custom Arm features chip foam sewn in directional channels to stimulate tissue and encourage the movement of lymph fluid. The garment is made using a moisture-wicking bacteriostatic material to keep the limb dry and free from infection.

### Ideal for

- Lymphedema
- Fibrotic tissue



*\*When combined with the oversleeve*



For a PDF of the product order form, scan the QR code or visit [sigvaris.group/mceforms](https://sigvaris.group/mceforms)

# Chipflow™

Variable Compression



## Gradient compression, high containment

The Chipflow pairs an inelastic wrap with a chip foam liner to form a garment that can be worn day or night. This garment is custom-sized to fit individuals with a small ankle and large calf, commonly seen in patients with lymphatic disorders.

Each Chipflow garment contains a channeled foam liner, which creates a pressure gradient that promotes lymph fluid movement and healthy skin.<sup>1</sup> It also contains a unique split-ankle zone that allows for independent application and maintains compression. The inelastic wrap is applied over the liner to allow for proper distribution of compression and a high level of containment.

Each Chipflow garment includes the Compreboot with gripper patches to allow for foot compression without slippage. Additionally, each garment includes non-compressive liners for protection of the skin.

This product is patent pending.

### Ideal for

- Lymphedema
- Edema
- Lipedema



# Chipflow™

## CALF & FOOT

The Chipflow Calf & Foot is an easy to apply garment made with comfortable, low profile, chip foam and an inelastic wrap. This combination provides gradient compression and high containment for the calf and foot.



# Chipflow™


## FULL LEG

The Chipflow Full Leg is an easy to apply garment made with comfortable, low profile, chip foam and an inelastic wrap. This combination provides gradient compression and high containment throughout the full leg.



### Night time or non-ambulatory time

Custom Chipflow is a breathable, easy-to-apply, garment for the management of lymphedema, lipedema, and general edema.

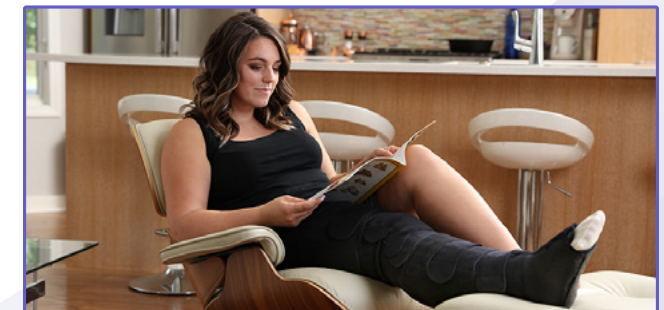

For a PDF of the product order form, scan the QR code or visit [sigvaris.group/mceforms](https://sigvaris.group/mceforms)

## Armassist®

30–40mmHg



The Armassist is a custom garment that offers firm, gradient compression with Flat Foam or Wavefoam. The device is constructed with a durable, medical grade fabric and includes adjustable straps. The medical grade fabric features non-static, non-absorbent, and non-stretch qualities as well as a palm pad for added dorsum compression.

### Ideal for

- Lymphedema



*Zipper panel along forearm for easy donning/doffing.*

## Legassist®

30–40mmHg



Sigvaris Legassist garments are ideal for patients with lymphedema. Constructed with a non-stretch medical grade fabric that is non-static and non-absorbent, the Legassist garments are manufactured with precision and a high standard of quality to the custom specifications provided by the clinician. This garment can be customized with Flat Foam or Wavefoam to promote the movement and drainage of lymph fluid.

### Ideal for

- Lymphedema



## Legassist® Standard CALF & FOOT

The Legassist Calf and Foot is a custom inelastic garment offering firm, gradient compression from the ankle to the knee bend. The product is low profile and lightweight, allowing for a comfortable fit. Clinicians may customize this garment further by selecting the preferred foam.



## Legassist® Custom Lobule Compression System


### LCS

The Legassist Lobule Compression System (LCS) is a custom compression garment used to lift and compress enlarged areas where lymph fluid has pooled. Patients with a lobule will benefit from the comfort and accurate fit the lightweight and conforming material provides. The Legassist LCS may be paired with the Legassist Calf and Foot. For added support and relief, select the optional Belt and Hip Attachment when ordering.



## Legassist® Full LEG

The Legassist Full Leg is a custom inelastic garment offering firm, gradient compression from the foot to gluteal fold. The product is low profile and lightweight, allowing for a comfortable fit. Clinicians may customize this garment further by selecting the preferred foam.

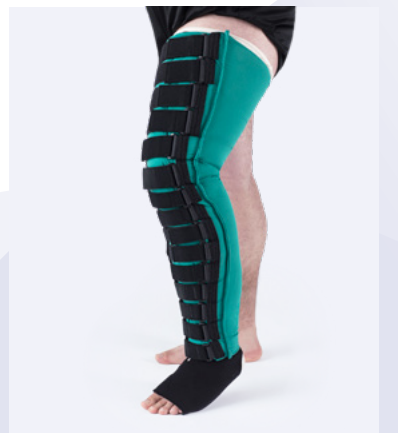

For a PDF of the product order form, scan the QR code or visit [sigvaris.group/mceforms](https://sigvaris.group/mceforms)

## Short Stretch Bandages

### BIAFORM

Case holds 20 rolls

Premium short stretch bandage with durability. Product offers superior stretch and conforming abilities.



Size	Qty
6cm x 5m	1 Case
8cm x 5m	1 Case
10cm x 5m	1 Case
12cm x 5m	1 Case
10cm x 10m	1 Case
12cm x 10m	1 Case

## Stockinette & Dressing Retainers

### COTTON STOCKINETTE

Ideal for use as an underliner with nighttime bandaging or compression garments. Cotton Stockinette is available in a range of sizes to fit almost any application. Product is non-compressive, 100% Cotton.



Size	Qty
3" x 25yds	1 Roll
4" x 25yds	1 Roll
5" x 25yds	1 Roll
6" x 25yds	1 Roll
8" x 25yds	1 Roll

### BIAGRIP

Compressive stockinette, which may be washed and reused to augment compression garments. Product may be cut to size.



Size	Qty
D 3" x 11yds	1 Roll
E 3.5" x 11yds	1 Roll
F 4" x 11yds	1 Roll
G 5" x 11yds	1 Roll
J 7" x 11yds	1 Roll
K 8" x 11yds	1 Roll

## Gauze & Tape

### ELASTOSOFT

Soft cotton elastic gauze. Ideal for wrapping fingers & toes.



Size	Qty
1" x 4.1yds	1 Bag / 24 Rolls
2" x 4.1yds	1 Bag / 12 Rolls
3" x 4.1yds	1 Bag / 12 Rolls
4" x 4.1yds	1 Bag / 12 Rolls

### DURAPORE®

Exceptionally durable, easy to cut to size surgical tape. Optimal for securing bandages.



Size	Qty
1" x 10yds	1 Box / 12 Rolls
2" x 10yds	1 Box / 6 Rolls

## Foam

### GRAYFOAM

Moderately rigid open-cell foam helps to soften fibrotic areas. The product provides firm, stable support under Biaform short stretch bandages.



Size	Qty
1/4"x18x72"	1 Sheet
1/2"x18x72"	1 Sheet

### WAVEFOAM

One sided

High-pressure ridges gently massage thickened fibrotic areas, while low-pressure channels allow pathways for fluid evacuation.



Size	Qty
1.2cm x 69cm x 90cm	1 Sheet

### WAVEFOAM II

Two sided

High-pressure ridges gently massage thickened fibrotic areas, while low-pressure channels allow pathways for fluid evacuation. Ridged or wave pattern is on both sides of the sheet.



Size	Qty
1.2cm x 69cm x 90cm	1 Sheet

## Padding

### BIASOFT

Case holds 10 rolls

Soft, durable, and seamless fleece padding for use under short stretch bandages. The product may be machine washed and dried for repeated use.



Size	Qty
6cm x 3m	1 Roll
6cm x 3m	1 Case
10cm x 3m	1 Roll
10cm x 3m	1 Case
15cm x 3m	1 Roll
15cm x 3m	1 Case

### NON-WOVEN

Soft Synthetic Padding  
Case holds 10 rolls

Optimal for use under short stretch bandages.



Size	Qty
10cm x 3.5m (4" x 3.8yds)	1 Case



## CHIP PAD DORSUM

Ideal for use on dorsum of hand or foot. Helps with softening scars from carpal tunnel surgery, or underneath compression bandages over pockets of lymphedema. Fits into Dorsal Pocket Glove.

Size	
Small	8cm x 9cm
Large	9cm x 14cm



## CHIP PAD AXILLA

Use under arm for treatment of lymphedema, especially over areas of post-operative scarring.

Size	
Small	10cm x 19cm
Large	18cm x 27cm



## CHIP PAD RADIAL HALF

Radial pathways create low pressure channels to promote fluid drainage. Use over effected areas or pockets of edema.

Size	
One size fits all	15cm x 25cm



## CHIP PAD RADIAL FULL

Versatile, effective design is ideal for post-surgical scarring and/or pockets of lymphedema.

Size	
Small	18cm diameter
Large	23cm diameter



## CHIP PAD BRA

Ideal for post mastectomy/lumpectomy edema. Pair with Comprest. Available in black or white.

Size
Half
Full



Chip Pad Bra Half



Chip Pad Bra Full

## CHIP PADS GENIFIT™


Ideal for use on dorsum of hand or foot. Helps with softening scars from carpal tunnel surgery, or underneath compression bandages over pockets of lymphedema. Fits into Dorsal Pocket Glove.

### Male



#### Tall

The rise of the tall male chip pad ranges in length from 32–44cm. See the measurement guide for reference.

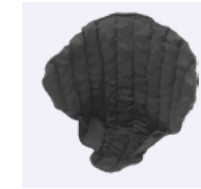

#### Short

The rise of the short male chip pad ranges in length from 14–22cm. See the measurement guide for reference.

Male Genifit					
	Small	Medium	Large	X Large	XX Large
Width	28cm	32cm	36cm	40cm	44cm
Scrotal Circ.	26cm	32cm	38cm	44cm	51cm
Rise – Short	14cm	16cm	18cm	20cm	22cm
Rise – Tall	32cm	35cm	38cm	41cm	44cm

### Female



#### Tall

The rise of the tall female chip pad ranges in length from 32–41cm. See the measurement guide for reference.



#### Short

The rise of the short female chip pad ranges in length from 14–20cm. See the measurement guide for reference.

Female Genifit				
	Small	Medium	Large	X Large
Width	28cm	32cm	36cm	40cm
Rise – Short	14cm	16cm	18cm	20cm
Rise – Tall	32cm	35cm	38cm	41cm

## ANKLE PAD

- Open-cell chip foam ankle padding
- Includes elastic stretch band to secure padding in place
- Constructed using moisture-wicking mesh lined fabric



Size
S/M/L

## FOAM LINER

- High density, open-cell foam with breathable, moisture-wicking mesh lining
- May be paired with inelastic wraps or worn separately



For a PDF of the product order form, scan the QR code or visit [sigvaris.group/mceforms](https://sigvaris.group/mceforms)



## COTTON SOCKS & LINER

*Non-compressive*

Soft, seamless cotton liners are non-compressive and can be worn under other compression products



Style	Size
Sock	S-XXL
Thigh High	S / M / L
Thigh High	XL
Arm	S-XL

## UNISTRAP

Neoprene wrap for added hand compression



Color
Black

## Accessories

Accessories and donning aids help patients manage their care by maximizing garment usage and providing additional comfort.

### Latex-Free Gloves

Gripping donning gloves recommended for use in a non-latex environment.



### Textile Gloves

These high-quality gloves have a latex surface to provide a good grip when putting on and taking off compression stockings.



### Washing Solution

Gentle washing solution specifically formulated to help maintain the quality of SIGVARIS GROUP compression socks and stockings. Also great for washing delicate garments!



### Sigvaris DOFF N' DONNER ROLLY®

A soft device that makes it simple to put on and remove all styles of graduated compression. It is perfect for both self-donning and caregiver use. Non-latex.



### Sigvaris DOFF N' DONNER CONE

This loading device makes it easy to roll stockings onto the Sigvaris DOFF N' DONNER. The bottom of the base firmly attaches to most clean, flat surfaces via suction.




### MELANY®

The donning and doffing device for compression stockings in case of limited mobility.



### Foot Slip

Comfortable applicator for use with open toe stockings.

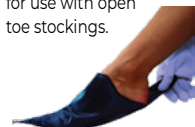

## For more information

We are available Monday–Friday from 8am–6pm EST.

Online: [sigvaris.com/en-ca](https://sigvaris.com/en-ca)

Call: **800.363.4999**



For a PDF of the product order form, scan the QR code or visit [sigvaris.group/mceforms](https://sigvaris.group/mceforms)

Notes

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

Notes

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

# SIGVARIS GROUP

## Customer Care

1-800-363-4999

## Email

canadaorders@sigvaris.com

## Find a retailer

sigvaris.com/en-ca/services/  
find-a-retailer

## SIGVARIS, inc.

8423 Chemin Dalton,  
Montréal, Qc H4T 1V5



SIGVARIS GROUP CANADA

Explore our download  
center for a copy of the **full  
catalogue and order form**



1. Data on file.

Durapore is a registered trademark of 3M.

Sigvaris, SIGVARIS GROUP, Accutab, Armassist, Chipvest, Compreflex, Comprofit, Compreboot, Compreshort, Compvest, Legassist, Medahand, Chipsleeve, Medaglove, and Medaform are registered trademarks of SIGVARIS AG, CH-9014 St.Gallen/Switzerland, in many countries worldwide.

Chipflow, Coolflex, Compreshape, Dorsal Pocket Glove, Genifit, Tension Tab, and 90° Flexi-Zone are trademarks of SIGVARIS AG, CH-9014 St.Gallen/Switzerland, in many countries worldwide. © Copyright 2024 SIGVARIS, Inc.

# sigvaris