

RESTORE ACCESS.™
PRESERVE OPTIONS.



Surfacer®
Inside-Out® Access



WHAT IF...

You could reliably and repeatedly **access the right internal jugular vein** and **avoid using other veins** which may compromise permanent arteriovenous access?

Introducing the Surfacer® Inside-Out® Access Catheter System, the proprietary Inside-Out® approach to restore access and preserve options.


In patients with venous occlusive disease, the current approach is to move to another vein if a subsequent central venous catheter (CVC) is needed. Failed venous access attempts may prevent permanent arteriovenous (AV) access, increasing patient morbidity and the overall cost of care.¹

To restore and preserve access in chronically occluded veins, John Gurley, M.D.² developed the Inside-Out® approach – allowing clinicians to change the standard of care.


A UNIQUE APPROACH



Totally occluded superior vena cava

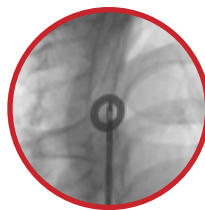

Exit target provides a zone to position the Surfacer® Device to exit the skin

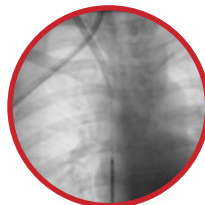

Introducer sheath locked onto needle wire and pulled into right atrium


RESTORE ACCESS.TM PRESERVE OPTIONS.

RESTORE ACCESS



- Achieves repeatable and reliable central venous access to the right internal jugular (RIJ) vein
- Preserves the viability of secondary central veins
- Optimizes placement and maturation of permanent AV access

RELIABLE PROCEDURE

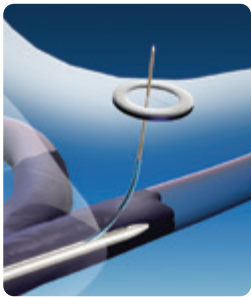

- 100% (12 of 12) of patients underwent successful central venous access placement³
- 100% (12 of 12) of patients maintained patency of central venous access through long-term follow-up visit³
- No device-related adverse events were reported within 48 hours and at 14-day follow-up visit³

REDUCE COSTS



- Supports the achievement of permanent AV access which reduces hemodialysis provider cost and downtime by decreasing catheter-associated morbidity and complications¹
- Easier, minimally invasive procedure compared to surgical bypass for patients with totally occluded central veins




CLINICAL EXPERIENCE

	Fluoroscopy time (min)	Technical success	Procedural complications
Inside-Out[®] Central Venous Access Clinical Experience⁴ (N=116)	4.1 ± 5.3	100%	1 (0.8%) ^b
Surfacer[®] System FIM Study³ (N=12 ^a)	7.4 ± 2.8	100%	1 (8.3%) ^c

REPEATED PROCEDURES

Percentage of repeated procedure counts on the same patient.
Clinical experience with the Inside-Out[®] procedure.⁴

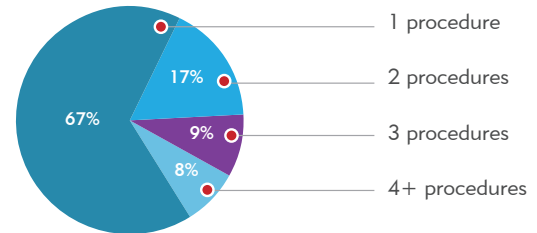


- a. All procedures completed with local anesthesia and light sedation.
b. Pocket hematoma due to excessive oral anticoagulation.
c. Mild hematoma at the clavicular exit due to tunneling and unrelated to the Surfacer device. Manual compression was not required.

REDUCE COSTS OF CARE¹


Achieving permanent AV access with the Surfacer[®] System

- ▶ Stop destruction of veins
- ▶ Improve quality of life

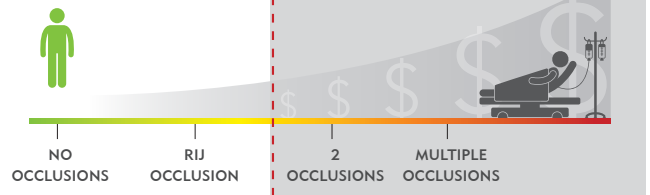

Surfacer[®]
Inside-Out Access

SURFACER® SYSTEM COMPONENTS

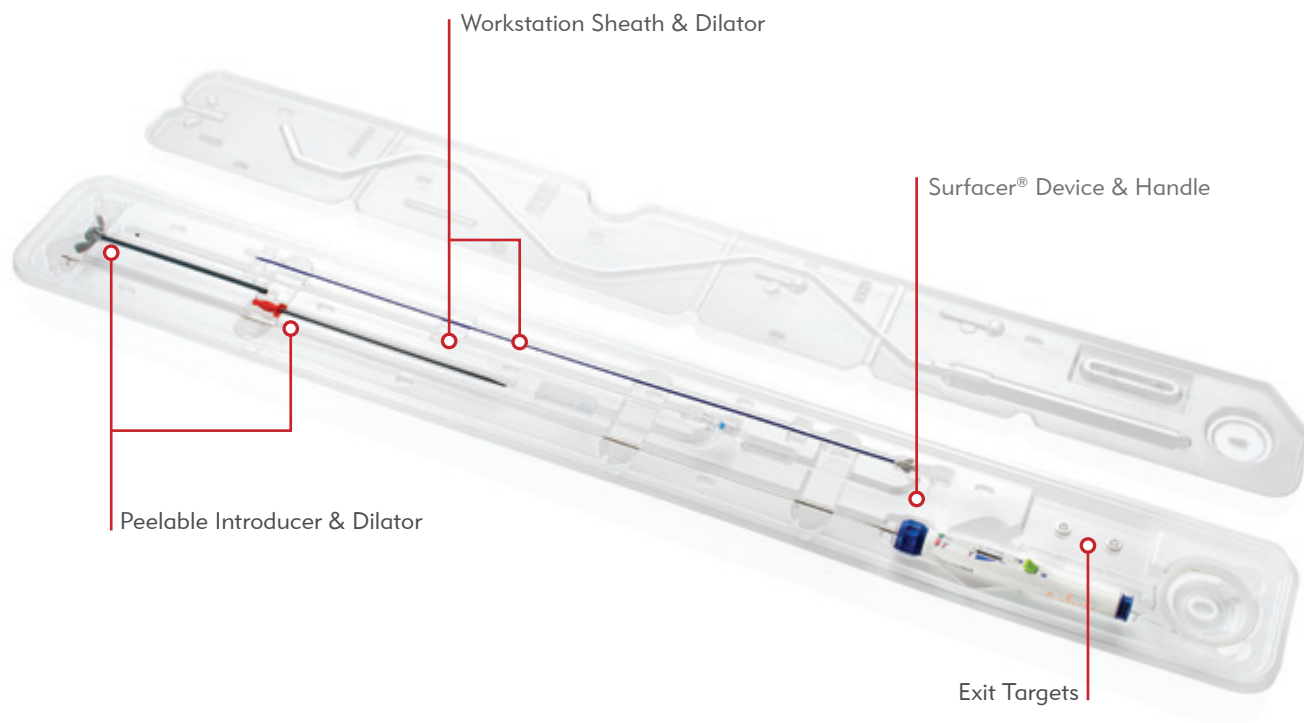

ORDERING INFORMATION

Catalog number	Description	Quantity
600200/C	Surfacer® Inside-Out® Access Catheter System (single)	1

REFERENCES

1. Data on file at Merit Medical.
2. Department of Cardiology, University of Kentucky, Lexington, KY, USA.
3. Ebner A, Gallo S, Cetraro C, Gurley J, Minarsch L. Inside-Out Upper Body Venous Access: The first-in-human experiences with a novel approach using the Surfacer Inside-Out Access Catheter System. Endovascular Today. June 2013.
4. John C Gurley, MD. Inside-Out Central Venous Access - Presentation (University of Kentucky May 9, 2012), Heart Rhythm Society 2012, May 9-12, Boston MA.



To Schedule Surfacer
Case Support, Please
Scan the QR Code

Before using refer to Instructions for Use for indications, contraindications, warnings, precautions, and directions for use.



Understand. Innovate. Deliver.™

merit.com

Merit Medical Systems, Inc.
1600 West Merit Parkway
South Jordan, Utah 84095
1.801.253.1600
1.800.35.MERIT

Merit Medical Europe, Middle
East & Africa (EMEA)
Amerikalaan 42, 6199 AE
Maastricht-Airport
The Netherlands
+31 43 358 82 22

Merit Medical Ireland Ltd.
Parkmore Business Park West
Galway, Ireland
+353 (0) 91 703 733